

# Anything They Handle Can Cross-Contaminate



**Everything They Handle Can Be Disinfected  
in 55 Seconds with The UV Box**

## AUVS Completes Your Hand Hygiene Bundle

No matter how high your Hand Hygiene compliance rate is, every handheld device your staff uses works against your ultimate goal. Add AUVS' KR615 Germicidal Enclosure - **The UV Box** - disinfect as many items as can fit in just 55 seconds and give your Hand Hygiene Program the support it needs!



**The UV Box** - Quickly becoming the standard in non-chemical disinfection of mobile handheld devices.

### Quick. Easy. Effective!

- Up to 4.5 Log, 99.99% Disinfection Rate for C. diff Spores According to Independent FDA Consulting Lab Protocol and Testing
- One-Button Simplicity: No Learning Curve
- Durable: No Moving Parts + Aluminum Case Protects UV Chamber
- 7 Year UV Lamp Guarantee
- Easy Lamp Replacement Access Panel
- Will Not Damage Valuable Equipment
- A Green Solution, Producing No Waster Products
- Our Patented Technology is Protecting the US Pentagon from Bio-Terrorism
- **Designed & Made in the USA**

# AUVS KR615 Germicidal Enclosures

Up to 4.5 Log, 99.99% Disinfection Rate in Less than 60 Seconds

Remember,  
every second they  
spend here...



...is a waste, if your  
Hand Hygiene Bundle  
doesn't disinfect these...



## One-Button Simplicity: No Learning Curve

- 1) Clean visible dirt, moisture or liquid residue from items to be disinfected as per manufacturer's instructions. In general, loose dirt can be cleaned with a soft duster or compressed air. Do Not scrub item or use paper towels such as hand towels from a sanitary facility.
- 2) Place item(s) in the KR615 Germicidal Enclosure.
- 3) For items with cords or leads that must remain connected, thread the cord through the unit's Wire Routing System.
- 4) When disinfecting multiple items do not stack or allow sides of inserted items to touch each other.
- 5) Close the door and turn knob to lock. Press the red button to begin the disinfection cycle.
- 6) When the red button and LED indicator turn off, remove item(s) and use normally.



**Note:** Covered surfaces not contacted by the UV light will not be disinfected.

## Two Models to Meet Your Disinfection Needs

	Medium KR615	Large KR615-L
Overall Dimensions	10" H x 7" D x 20" W	10" H x 10" D x 20" W
Enclosure Dimensions	3.5" H x 6" D x 15.5" W	3.5" H x 8" D x 15.5" W
Weight	10 lbs.	10 lbs.
Power	110V, AC	110V, AC
Bulb Life	7 Years with Heavy Use	7 Years with Heavy Use