


UV-BOX-E2/40H-NX-T

UV BOX E		2/40H-NX-T
LAMP LIFETIME (hour)*	DURATA LAMPADA (ore)*	≤ 18.000
RECOMMENDED REPLACEMENT	RICAMBIO CONSIGLIATO	(Maximum Every 2 Years) (Max Ogni 2 Anni)
TOTAL CONSUMPTION (W)	CONSUMO TOTALE (W)	80
EXTERNAL DIMENSIONS HxLxP (mm)	DIMENSIONI ESTERNE HxLxP (mm)	129 x 509 x 275
INTERNAL DIMENSIONS HXLXP (MM)	DIMENSIONI VANO INTERNO HxLxP (mm)	89 x 469 x 213
WEIGHT (Kg)	PESO (Kg)	5.5
INOX SHELVES	RIPIANI IN ACCIAIO INOX	1
PROTECTION RATING	GRADO DI PROTEZIONE	IP 20
REPLACEMENT LAMP	LAMPADA UV DI RICAMBIO	n°2 CHS-40WH

* continuous operation/accensione continuativa

TECHNICAL DRAWINGS

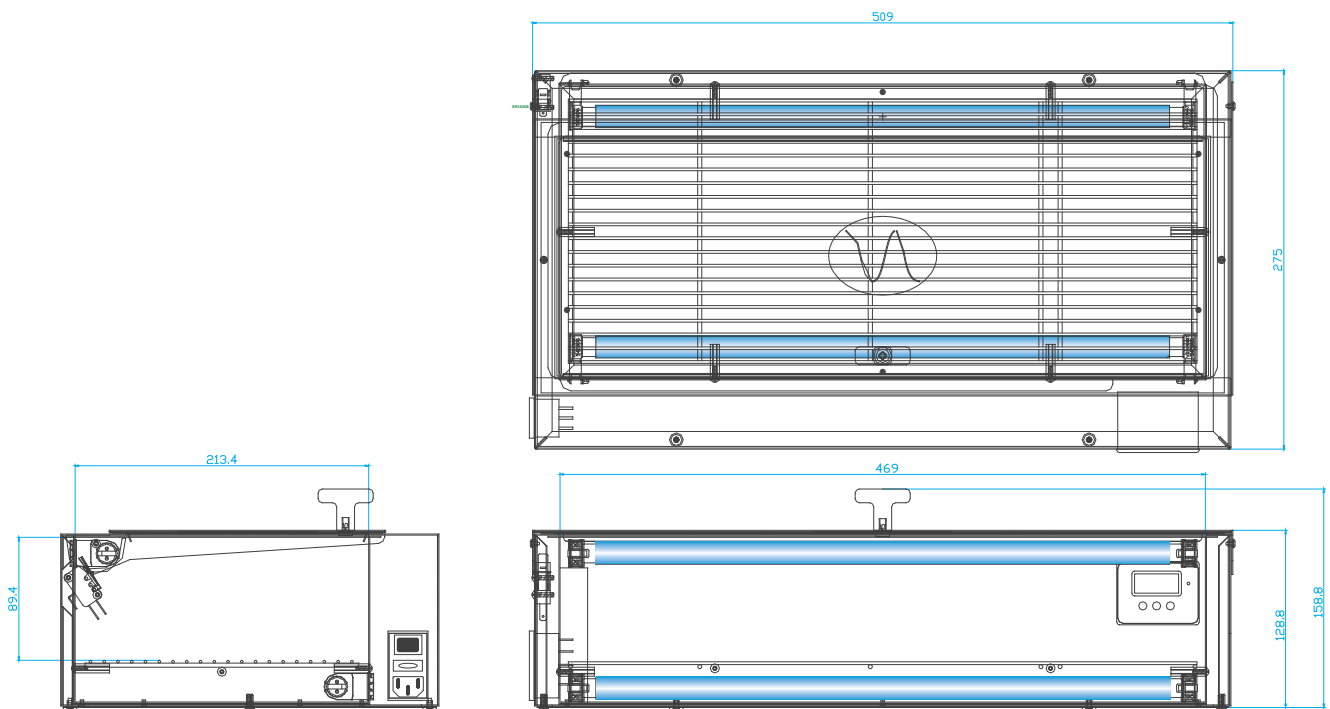


INSTALLATION

UV-BOX-E2/40H-NX-T is a compact desktop unit. After placing it in position, insert the plug.

- 1) Switch on the unit via the ON/OFF switch on the left side (with green indicator light).
- 2) Set the desired switch-on time using the digital timer (see picture 1).
- 3) When the lid is closed, the UV-C lamps will turn on, the correct operation will be visible by the wave-shaped cut made on the lid (see picture 2)
- 4) In any case, when the cover is opened, the lamps will be switched off for the operator's safety.

UV-BOX-E2/40H-NX-T è un apparecchio compatto da tavolo. Dopo averlo posizionato inserire la spina.

- 1) Dare tensione all'apparecchio mediante l'interruttore ON/OFF posizionato sul fianco sinistro (con spia verde di segnalazione).
- 2) Impostare il tempo di accensione desiderato utilizzando il timer digitale (vedi immagine 1)
- 3) Alla chiusura del coperchio si attiverà l'accensione delle lampade UV-C, il corretto funzionamento sarà visibile dal taglio a forma di onda realizzato sul coperchio (vedi immagine 2)
- 4) In ogni caso all'apertura del coperchio, si spegneranno le lampade per la sicurezza dell'operatore.

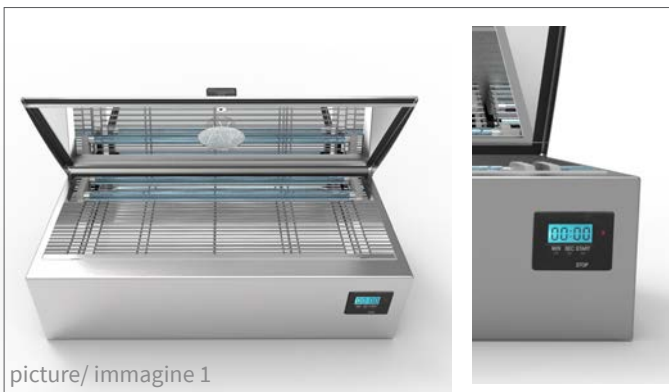

picture/ immagine 1



picture/ immagine 2

CODES

